ULTRA Covi-Catch™

Covid-19 Rapid Antigen self-test quick guide

Use Nasal Swab

Before performing the test, read complete quick guide properly.

OVERVIEW AND DESCRIPTION

Corona virus is a single-stranded positive-sense RNA virus with an envelope of about 80 to 120 nm in diameter. Its genetic material is the largest of all RNA viruses and is an important pathogen of many domestic animals, pets, and human diseases. It can cause a variety of acute and chronic diseases. Common signs of a person infected with a corona virus include respiratory symptoms, fever, cough, shortness of breath, and dyspnea. In more severe cases, infection can cause pneumonia, severe acute respiratory syndrome, kidney failure, and even death. The 2019 new corona virus, or "SARS-CoV-2 (COVID-19)", was discovered because of Wuhan Viral Pneumonia cases in 2019, and was named by the World Health Organization on January 12, 2020, confirming that it can cause colds and the Middle East Respiratory Syndrome (MERS) and more serious diseases such as acute respiratory syndrome (SARS). These kits are helpful for the auxiliary diagnosis of corona virus infection. The test results are for clinical reference only and cannot be used alone as a basis for confirming or excluding cases.

PURPOSE OF USE

Individuals with symptoms of respiratory infection are tested for the presence of COVID-19 virus antigens by immunochromatography in a nasal specimen. It is an *in vitro* diagnostic medical device. Asymptomatic individuals are not advised to undertake the home test.


TEST PRINCIPLE

ULTRA Covi-Catch Test has two pre-coated lines, Control line (C) and Test line (T) on the surface of the nitrocellulose membrane. Both the control line and test line in the result window are not visible before applying any specimen. During the test, Covid-19 antigen in the specimen interacts with monoclonal gold conjugate making antigen-antibody color particle complex. This complex migrates on the membrane via capillary action until the test line, where it will be captured by highly specific antibody. A colored test line would be visible in the result window if Covid-19 antigens are present in the specimen. The intensity of colored test line will vary depending upon the amount of Covid-19 antigen present in the specimen. If Covid-19 antigens are not present in the specimen, then no color appears in the test line. The control line is used for procedural control, and should always appear if the test procedure is performed properly and the test reagents are working.

KIT STORAGE AND STABILITY

Store the test kit at room temperature, 2-40°C out of direct sunlight. Materials provided are stable until the expiry date printed on the box. DO NOT FREEZE.

Kit Content

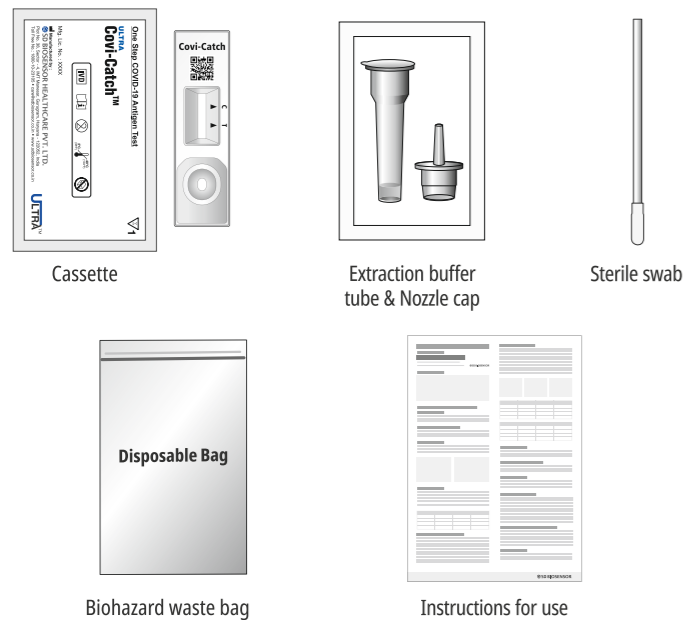
STEP 1 Preparation for test

1 Wash your hands thoroughly before testing.


! When using the product, wear disposable plastic gloves, etc.

Scan the QR Code on the package to download the Covi-Catch App or Download the Covi-Catch App from Store.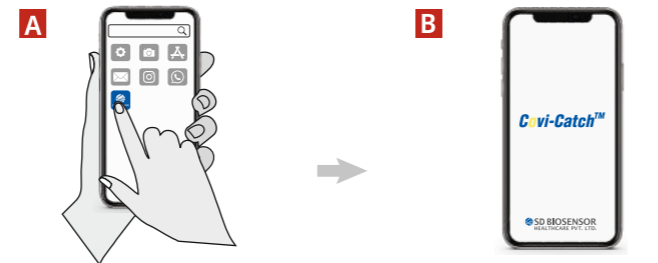

Fill all the details in the app. All fields are mandatory.

Check the expiry date at the back of the cassette package.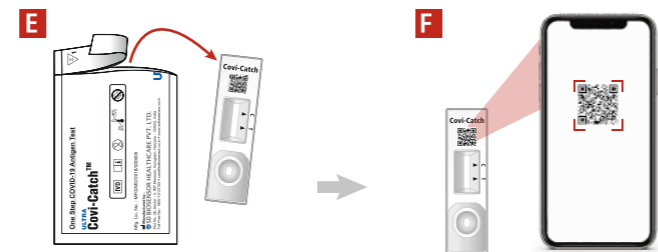


Open the cassette package to take out the cassette.

Scan the QR code on the cassette to link the code with your credentials.


! Perform the test immediately after opening the cassette package.

2 Open the extraction buffer tube & nozzle cap pouch. Remove the lid of the extraction buffer tube. Place the extraction buffer tube into the extraction buffer tube holder in the package.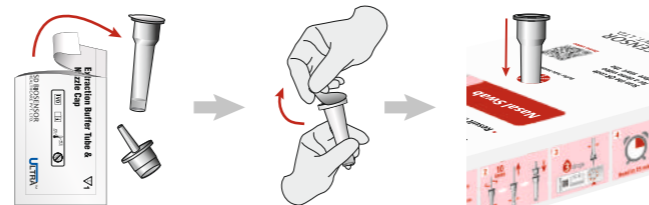


STEP 2 Specimen collection

3 Open the sterile swab pouch and take it out.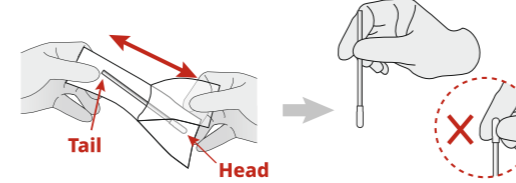


! Be careful not to touch the swab head.

4 Insert the same sterile swab up to 2 cm in both nostrils one by one and rub at least 5 times.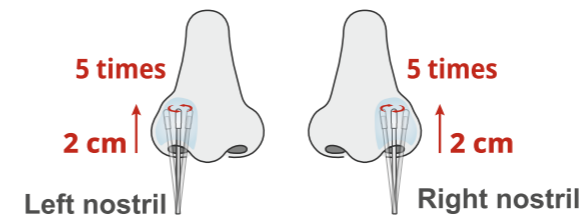


STEP 3 Test procedure

5 Put the swab removed from inside of the nose into the extraction buffer tube. Stir it for 10 times or more. Remove the swab while squeezing the sides of the tube to extract the liquid from the swab and then close the nozzle cap by pressing it.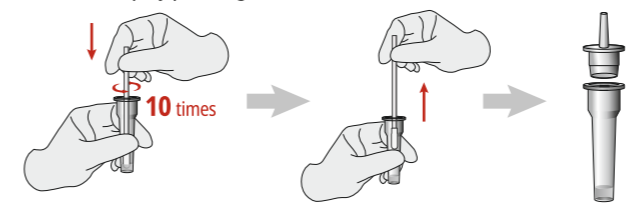


! Do not swallow or allow the solution to come into contact with your eyes or wound. Keep it away from the reach of children.

6 Apply 3 drops of extracted specimen to the specimen well of the cassette.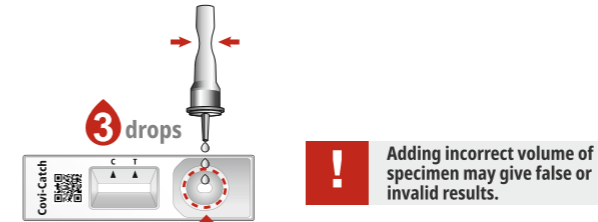

After adding the specimen, start the timer in the app.

7 The mobile phone will give an alarm at the end of 15 minutes. Click the picture of the test card at 15 minutes.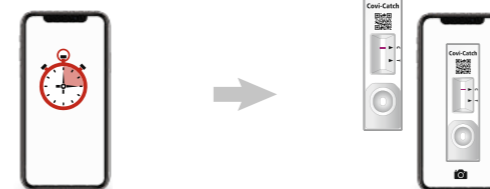

! Do not read the test result after 30 minutes. It may give false results.


Wait for the App to analyse and display your Covid-19 test result.

STEP 4 Interpretation of the test result

Positive Result

If both the Control line "C" and the Test line "T" is visible, the result is Positive. Even if the test band is faint, it should be considered as positive.

Note: All Individuals who test positive may be considered as true positives and no repeat testing is required. All positive individuals are advised to follow home isolation & care as per the ICMR & Ministry of Health & Family Welfare (MoH&FW) protocol which can be accessed at: <https://www.icmr.gov.in/chomecare.html>

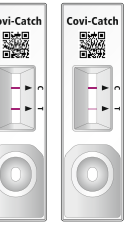Negative Result

If only control line "C" and no test line "T" is visible, the result is Negative.

Note: All symptomatic individuals who test negative by Rapid antigen test should get themselves tested by RT-PCR. This is especially important as the Rapid antigen test are likely to miss few positive cases presenting with a low viral load. All Rapid antigen test negative symptomatic individuals may be treated as suspect COVID-19 cases and are advised to follow ICMR/MoH&FW home isolation protocol while awaiting the RT-PCR test result.


Invalid Result

If the control line "C" is not visible, the result will be invalid. Retesting is needed with a new cassette in such cases by following all the steps again.

STEP 5 Discarding the used components

The swab inserted in the Extraction buffer tube kills SARS-CoV-2 & other viruses. All used components of the test should be put inside the waste disposal bag and thrown in the household waste. Disinfect all surfaces that the specimen may have touched and wash your hands after disposal.

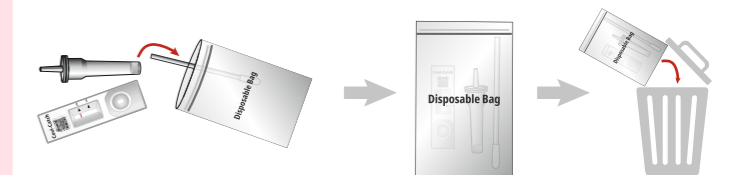All trademarks referred are trademarks of either SD BIOSENSOR HEALTHCARE PVT. LTD. or their respective owners. © 2021 SD BIOSENSOR HEALTHCARE PVT. LTD. All rights reserved.

Issue date : 2021.10
 Manufactured by
SD BIOSENSOR HEALTHCARE PVT. LTD.
 Manufacturing site: Plot No. 38, Sector - 4, IMT Manesar, Gurugram, Haryana - 122052, India
 Registered office: Unit No - 202 A-D, 2nd Floor, Tower A, Unitech Signature Towers, South City 1, Gurugram, Haryana - 122001, India
 Any inquiries regarding the instruction provided should be addressed to: care@sdbiosensor.co.in or call at - **1800-10-23105**
www.sdbiosensor.co.in

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